



Single Patient Use Linear Stapler and Subassembly

Availability of Yellow Subassembly Cartridge Reload

- White, blue, yellow & green cartridge reloads satisfy stapling of various tissue thickness
- 45mm Linear Stapler - the prioritized option of rectal surgery



Major Features

- Closure of GI tract residual defect
- Suturing of GI tract incision
- Suturing & Cutting of Lung
- Composition of double stapling technique

Golden Stapler Surgical Co., Ltd.

CE ISO SFDA

0434 9001/13485:2003

Advancing Surgical Technology
to Benefit More People

Single Patient Use Linear Stapler and Subassembly

Positioning Pin

Avoid slippage of clamped tissue
Assist parallel closure
Ensure complete suturing of targeted tissue



Parallel Closure

Even compression on targeted tissue
Provide uniform formation height of all staples
Effective suturing & homeostasis



Intermediate Position

Adjust suturing position on targeted tissue
Ensure suturing on targeted position



Separated Closure & Firing Handles

Clear surgical procedures
Safe & easy to use

Closure Handle
Firing Handle



Blue, Yellow & Green Cartridge Reloads

Satisfy suturing of various tissue thickness
Effective suturing & homeostasis



3 Different Sizes of 30, 45 & 60

Satisfy suturing of various GI tract width
Economic & simple using



Tactile & Audible Responses

Indication of safety & effectiveness in surgery



Code	Colour	Staple Rows	Staple Qty	Closed Staple Height (MM)	Suturing Length (MM)	Applicable Tissue Thickness	Code of Subassembly Cartridge Reload
LSC30W	White	3	23	1.0	30	Blood vessel, thin tissue	LSCR30W
LSC30B	Blue	2	11	1.5	30	Normal thick tissue	LSCR30B
LSC30G	Green	2	11	2.0	30	Thick tissue	LSCR30G
LSC45B	Blue	2	15	1.5	45	Normal thick tissue	LSCR45B
LSC45Y	Yellow	2	15	1.8	45	Medium thick tissue	LSCR45Y
LSC45G	Green	2	15	2.0	45	Thick tissue	LSCR45G
LSC60B	Blue	2	21	1.5	60	Normal thick tissue	LSCR60B
LSC60Y	Yellow	2	21	1.8	60	Medium thick tissue	LSCR60Y
LSC60G	Green	2	21	2.0	60	Thick tissue	LSCR60G

Golden Stapler Surgical Co., Ltd.

Add: Building 7A, Hutang Kechuangyuna 256 Mingxin Zhong Road,
Hutang Town Wujin, Changzhou, Jiangsu, China

Tel: +86 519 85172266

Fax: +86 519 85172299

Web: <http://www.goldenstapler.com>

P.C: 213164